



## Marena Reusable Antibacterial Face Mask for Children\*

### Style: **CHILDMASK**

Washable, reusable fabric face mask with active silver antibacterial protection to help guard face, nose and mouth against the spread of respiratory droplets and splashes.

This mask helps protect against the spread of larger respiratory droplets, germ particles, and splashes. While this mask will provide some protection against, it will not act as a barrier against the spread of smaller respiratory droplets or germ particles.

- Suited for 5-16 year olds, depending on measurements, with adult supervision.
- Washable up to 75 times without losing effectiveness.
- Soft, skin-friendly TriFlex™ fabric with 3D Stretch for optimal comfort. The clean-finished, soft edge under the eyes does not leave an imprint on the skin surface.
- Lightweight, one-piece design covers the nose and mouth, with comfortable ear straps to ensure the mask stays in place.
- Good airflow and breathability.
- Cost per use significantly lower than disposable masks.
- Available in sizes small, medium, and large.



#### HOW TO PUT ON:

When putting the mask on, place shiny side to face. The contrasting color seam, known as a bar tack should be near the chin. Put loops over ears, and adjust to tightly cover the nose and mouth. If a tighter fit is required, the loops may be twisted before placing over ears.



#### HOW TO SAFELY REMOVE:


To remove the mask, avoid touching outer layer of mask or face. Remove from ears, only touching loops, and immediately put into washer, sterilizer or safe storage. It is best performed under adult supervision. Avoid putting the face mask around neck or forehead.




#### HOW TO CLEAN:

Clean mask by washing or sterilizing before each use. Sanitize hands immediately after handling worn masks each time. Clean after multiple use or if soiled. Discard if damaged or torn. See reverse for garment care instructions.

### The Mask Size is Indicated by Seam Colors

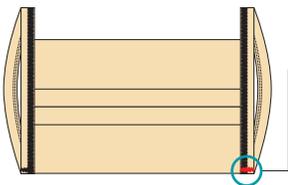

The size of the mask is indicated by a color coded seam, called a bar tack, located at the bottom of the mask where the two ear loops are attached.

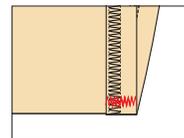

-  **SMALL** | Green bar tack
-  **MEDIUM** | Red bar tack
-  **LARGE** | Yellow bar tack

For more information on our Face Mask or to order, visit [www.marena.com/pages/child-mask-info-reorder](http://www.marena.com/pages/child-mask-info-reorder)



## Tips to encourage a child to wear a face mask:

Wearing a face mask is a new experience for everyone. It may be especially frightening for kids. Here are a few tips to encourage wearing and proper handling of the masks.

- 1. Talk about it.** Empower them by sharing how wearing a face mask gets them involved in the fight against COVID-19. Remember, wearing a face mask helps to slow the spread of the virus by limiting the transmission of large respiratory droplets, germs and splashes.
- 2. Make sure you are wearing a face mask too!** This will let your child know they are not alone and that you too are doing your part to slow the spread of the virus
- 3. Put a face mask on their favorite stuffed animal or toy.** Share photos of other kids wearing a mask. Showing kids like them or having their favorite toy wear a face mask will help to normalize this change.
- 4. Remind them not to touch their masks.** Keeping their hands away from their face will help prevent the transfer of germs from hard surfaces to their masks.
- 5. Washing hands is key to slowing the spread.** Encourage them to wash their hands before applying and removing the mask.

For additional tips and resources for talking to kids about COVID-19 and wearing a mask, visit [www.marena.com/page/child-mask-info-reorder](http://www.marena.com/page/child-mask-info-reorder) or scan this QR code.



## How to Reorder

Reordering the Marena Reusable Antibacterial Face Mask for Children is easy! You may reorder by phone, email, or the web. When ordering, share the color of your bar tack seam to indicate sizing.

Online: [www.marena.com/pages/child-mask-info-reorder](http://www.marena.com/pages/child-mask-info-reorder)

By Phone: +1-888-462-7362

Email: [orders@marenagroup.com](mailto:orders@marenagroup.com)

## Garment Care & Cleaning



Machine wash; tumble dry. Do not use fabric softener. Do not use bleach

This product has been tested to be effective for up to 75 washes. May be washed at temperatures: >160F (>71 C).

May be steam cleaned at temperatures between 212°F (100°C) and 270°F (132°C).

Ironing is permitted at temperatures up to 365°F (185°C).

Product related issues should be reported to the manufacturer as well as the appropriate regulatory body.

\*This product is authorized for sale under a US FDA Emergency Use Authorization (dated April 18 and 24, 2020) as a Non-Surgical Face Mask (Class I medical device) for use by the general public in non-healthcare settings, and as source control in healthcare settings to help prevent the spread of infection or illness during the COVID-19 pandemic. This product is non-sterile and has not been tested to N95 or other standards for particulate filtration, liquid barrier protection, or infection prevention. It is not intended to be used as a respirator or in such clinical setting where the risk level of infection through fluids or inhalation exposure is high. This mask has not been tested for effectiveness against the COVID-19 virus. Each mask is intended for use by one person only and must not be shared. This mask is prohibited for anyone who has trouble breathing, is unconscious, incapacitated or otherwise unable to remove the mask without assistance. The mask is not a toy and must only be used and/or cleaned under adult supervision. It is not intended for use by children under five years old and should not be worn by a sleeping child. Please discard packaging after opening as the plastic packaging may cause suffocation if placed over head. See instructions for use and cleaning, provided separately.