

FERNDALE SCHOOL DISTRICT NO. 502
 Ferndale, WA 98248

District AED Checklist

Location: _____ Unit: LIFEPAK 500 Serial No. _____

Instruction	Recommended Corrective Action	C	H	E	C	K		B	O	X	E	S	
		Jan	Feb	Mar	Ap	May	June	July	Aug	Sept	Oct	Nov	Dec
Examine the AED case, connector, and battery well for: Foreign Substances Damage of Cracks	Clean the device Contact Sp Services for service												
Examine the battery pins for bending or discoloration.	Discard and replace battery												
Check expiration date on batteries and therapy electrodes.	Replace if expired												
Examine the accessory cables for cracked, damaged, broken or bent connectors or wires.	Replace damaged or broken parts												
With the battery installed, press on/off to turn on the AED and look for: Self test messages Momentary illumination of each LED and LCD segments Battery low or replace battery Service indicators or call service message	If absent, contact authorized service personnel. If absent, contact Sp. Services to repair or replace parts. Replace battery immediately. Contact authorized service personnel.												

Unit Inspected by:

Date													
Initial													
Service needs reported to Special Services Date/Time/Contact person													

_____/_____
 Staff Signature Initials Staff Signature Initials Staff Signature Initials