

# Roger Touchscreen Mic with Roger Focus II

## Roger Focus



This part must be worn on the ear. RED is RIGHT, BLUE is LEFT.

**Power ON** (and OFF at end of day) by holding down the bottom end of the toggle switch.

**Charge** every night. Turn the Roger Focus OFF and place behind the ear portion in the charging ports. In the morning, remove and turn them ON.

## Roger Touchscreen Mic



This is the portion that will be passed from teacher to teacher.

**ON/OFF** switch is on the right side—press and hold for a few seconds.

**MUTE** button is on the front bottom. Microphone should be muted when student is doing independent work, teacher is leaving classroom, teacher is having a private conversation, etc.

**CONNECT** button (touchscreen option) can be used if the Roger Focuses become disconnected



4 ways to use!



**Lecture Mode:** For direct instruction: Teacher wears on lanyard 6-8 inches below mouth

**Small Group Mode:** Place flat on table to pick up voices of group

**Pointing Mode:** For class skits/presentations/assemblies: Teacher or student can point to the person speaking within 20 feet; good for class skits/presentations when multiple people are speaking

**Computer/Media Listening:** Plug auxiliary cable into the microphone and into the laptop/chromebook to replace headphones



**Charging:** Must be charged every night. Turn the Roger Focus OFF and place behind the ear portion in the charging ports. In the morning, remove and turn them ON.

