

# Attaching Audioshoes and Receivers to Phonak Hearing Aids

Scan code for video instructions



**To Attach the Audioshoe:** *Do this every morning.*



**Step 1:** Match the 3 prongs on the receiver to the 3 prongs on the audioshoe. Push until you hear a click



**Step 2:** Note the track mechanism on the bottom of the hearing aid and audioshoe



**Step 3:** Slide the audioshoe onto the hearing aid by aligning the track mechanism



**Step 4:** When it is connected, you'll hear a click

**To Detach the Audioshoe:** *Do this before the student goes home.*



**Step 1:** Push down on the button with your fingernail



**Step 2:** While pushing the button, slide the audioshoe off of the hearing aid



**Step 3:** Store in a safe place until the student puts it back on!