

AED REPORTING

MONTHLY AED checks are required by the 7th of each month.



You don't need it until you need it.

Make sure your building is ready.

If you have a different model please notify CPR@dallasisd.org

ZOLL PLUS



Ensure lid is securely shut/closed.



AED unit is operational



AED unit is not operational; notify CPR@dallasisd.org immediately


Adult pad is located inside and connected to AED. Yellow seal tape contains exp date

Separate Pedi pad



Ensure the pedi package remains **sealed** and located in the soft black AED carry bag.

ZOLL 3

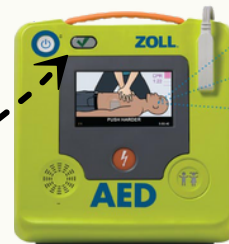

✓ AED unit is operational

No ✓ means AED unit is not operational; notify CPR@dallasisd.org immediately

This unit is equipped with **one** universal electrode pad that can be used on pedi and adults. Located in a slot on the back of the unit. Verify package is sealed and plugged into front of unit.



Electrode pad date



located either on yellow seal tape or on sticker found on pad package.

DO NOT remove the AED lid or break the seal/tape unless ready to use the AED unit

broken/missing yellow seal tape



Please notify CPR@dallasisd.org immediately

When an AED is used in a medical event, call Health Services (972) 925-3386.