

ZOLL PLUS AED HOW-TO REPLACE ADULT PADS

01.

Open the Lid

Pop the lid off the AED unit by lifting it up from the front. The yellow expiration tab on the pads is typically threaded through a slit in the lid.

02.

Remove Old Pads

Hold the defibrillator down with one hand for stability, and with the other, pull up on the old pad's cable connector to disconnect it from the port in the machine.

Cut the wire and throw old pads in regular trash can

03.

Insert New Pads

Plug the new pad's cable into the port on the AED unit. The connector is designed to fit only one way.

- Make sure the pictures on the new pads are facing down, aligning with the corresponding pictures on the defibrillator unit itself.

04.

Secure Cables & Close Lid

1. Make sure all excess cables are tucked into the compartment.
 - Thread the yellow "convenience tab" (which shows the new expiration date) through the slit in the top of the lid.
 - Place the back of the lid on first, then push the lid down firmly until you hear it **snap** into place.

05.

Verify Readiness

The AED will perform an automatic self-test once the new pads are connected and the lid is closed.

- Check the status indicator window on the front of the unit. A **green check mark** will appear, confirming the unit is ready for rescue.

[Click here to watch step by step video](#)